

Suggestions for performance evaluation

with the **LIFEPAK® 20/20e** Defibrillator/Monitor

Maintenance and reports

Name: _____ Unit: _____

Reviewer: _____ Date: _____

This Performance Evaluation is a suggested assessment of the user's ability to maintain and obtain reports from the LIFEPAK 20/20e defibrillator/monitor series. This evaluation does not cover all information and skills required to operate the device safely and effectively. This evaluation is designed to be completed after observing an equipment demonstration given by a qualified instructor. For complete information, review the Operating Instructions. References to buttons are indicated in **Bold** and screen display messages are indicated in *Italics*.

Performance criteria	Complete	Incomplete	Comments
<p>1. User test:</p> <p>a. Presses ON.</p> <p>Note: To perform the user test, ignore all REMOVE TEST PLUG prompts and keep the test plug connected.</p> <p>b. Presses OPTIONS. If the defibrillator is in AED mode, switch to manual mode and then press OPTIONS.</p> <p>c. Selects <i>USER TEST</i>.</p> <p>d. Selects <i>YES</i> to initiate user test.</p>			
<p>2. Therapy cable test:</p> <p>a. Confirms therapy cable is connected to the defibrillator.</p> <p>b. If QUIK-COMBO therapy cable is connected:</p> <ul style="list-style-type: none"> • Confirms test plug is connected to therapy cable. • Presses ANALYZE, look for the REMOVE TEST PLUG message. <p>c. If hard paddles are connected:</p> <ul style="list-style-type: none"> • Confirms paddles are properly seated in wells. • Selects 10J and presses CHARGE button. • When fully charged presses SHOCK button on paddles, and look for <i>ENERGY DELIVERED</i> message. <p>Note: Inspect defibrillator and accessories per LIFEPAK 20/20e defibrillator/monitor Operator's Checklist located in Appendix D of the operating instructions.</p>			
<p>3. Changing recording paper:</p> <p>a. Presses the black button to open the printer door.</p> <p>b. Inserts the new paper roll onto raised spool brackets with grid facing forward.</p> <p>c. Pulls out a short length of paper.</p> <p>d. Pushes the printer door in to close.</p>			
<p>4. Entering patient data:</p> <p>a. Presses OPTIONS and selects <i>PATIENT</i>.</p> <p>b. To enter or edit a patient's name or ID: Select LAST NAME, FIRST NAME, or ID. Rotate the Speed Dial to select the desired character.</p> <p>c. Selects the letters and symbols identifying the patient's name, ID, etc</p>			

