

Hunter Tendon Implants

Quick reference guide

Overview

The Hunter Tendon Implants are indicated for use in stage one of the two-stage procedure for the reconstruction of the flexor and extensor tendons in individuals having significant hand tendon injury.

Stage 1

Rods are implanted temporarily to replicate the natural tendon allowing a pseudosynovial sheath to form which is intended to later nourish and lubricate an autogenous tendon graft.

Stage 2

The time between implantation and removal of the device is typically 2 to 6 months to permit maturation of the tendon bed to the point where it can nourish and lubricate a tendon graft. The Palmaris tendon is favored by hand surgeons for the tendon graft. The Plantaris tendon can also be used.

Swanson

Spacer (Passive)



- Passive Only
- 100% Silicone Elastomer
- Sizing Set Available
- Elliptical Shape

Hunter

Standard (Passive)



Passive (Passive)



Active (Active)



- Barium Treated Silicone Elastomer with a woven polyester core
- Passive and Active require a stainless steel 2.0mm bi-cortical screw (not provided), fixation with twisted wire can also be utilized
- No Sizing Set

Active vs. Passive*

Active	Passive
Patient's finger has ability to move postoperatively.	Patient's finger does not move postoperatively.
Pseudosynovial sheath forms while patient's finger is active.	Pseudosynovial sheath forms while patient's finger is passive.
Proximal attachment to select motor tendon: Profundus motor is first preference, Superficialis is second.	Highest sold product codes.

*Ultimately based on surgeon preference or previous training.

Ordering information



Hunter Standard Tendon Implant 24.5cm Length

Reference	Description
TR200000	2mm Wide
TR300000	3mm Wide
TR400000	4mm Wide
TR500000	5mm Wide
TR600000	6mm Wide



Hunter Passive Tendon Implant 25cm Length

Reference	Description
PT300000	3 mm Wide
PT400000	4 mm Wide
PT500000	5 mm Wide
PT600000	6 mm Wide



Hunter Active Tendon Implant 4mm Width

Reference	Description
ATPC1640	16cm Long
ATPC1840	18cm Long
ATPC2040	20cm Long
ATPC2240	22cm Long
ATPC2640	26cm Long



Swanson Tendon Spacer

Reference	Description
24270003	24cm x 3mm
24270004	24cm x 4mm
24270005	24cm x 5mm
24270006	24cm x 6mm
24370001	Swanson Sizers (Provided Non-Sterile) (Green=3, Blue=4, Yellow=5, Red=6)

This document is intended solely for the use of healthcare professionals. A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that surgeons be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate the breadth of Stryker's product offerings. A surgeon must always refer to the package insert, product label and/or instructions for use, including the instructions for cleaning and sterilization (if applicable), before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Hunter, Stryker. All other trademarks are trademarks of their respective owners or holders.

Content ID: **AP-013170A_05-2022**
Copyright © 2022 Stryker



Manufactured by:

Wright Medical Technology, Inc.
1023 Cherry Road
Memphis, TN 38117

161 Rue Lavoisier
38330 Montbonnot Saint Martin
France

stryker.com